

# Contents

---

Preface	4
Evaluate potential contraindications	7
Step 1: Evaluate potential contraindications	7
Prepare the patient for EIT monitoring	8
Step 2: Apply measurement tape	8
Step 3: Measure patient	9
Step 4: Determine the SensorBelt size	10
Step 5: Choose the SensorBelt	11
Step 6: Prepare the foam (ContactAgent)	12
Step 7: Apply the foam	13
Step 8: Position the SensorBelt	14
Step 9: Fit the SensorBelt (1)	15
Step 10: Fit the SensorBelt (2)	16
Step 11: Secure the SensorBelt	17
Step 12: Attach the SensorBelt (1)	18
Step 13: Attach the SensorBelt (2)	19
Step 14: Start up the SensorBelt	20
Step 15: Indicator light on SensorBelt	21
Configuration of the intensive-care ventilator elisa 800 <sup>VIT</sup>	22
Step 16: Activate EIT	22
Step 17: Enter patient data	23
Step 18: Minimize sources of interference	24
Step 19: Check the settings and the SensorBelt	25
Evaluate the results	26
Step 20: Evaluate regional ventilation distribution	26
Step 21: Evaluate unequal ventilation distribution between the two lungs	27
Step 22: Analyse SilentSpaces: Gravity-independent form	28

---

Step 23: Analyse SilentSpaces: Gravity-dependent form	29
Step 24: Analyse SilentSpaces: Bilateral form	30
<b>Terminate EIT monitoring</b>	<b>31</b>
<b>Final step: Terminate EIT monitoring</b>	<b>31</b>
<b>FAQ: Practical Q&amp;A</b>	<b>32</b>
Contraindications	32
Potential complications and sources of error	32
SensorBelt	34
Interpretation	35
<b>Core statements of the TREND group</b>	<b>37</b>
<b>Selected studies</b>	<b>48</b>
Basics	48
Optimising ventilation settings	48
Display of regional lung function	49
Patient positioning	50
EIT validation	50